ENGLISH
BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.

INTENDED USE/INDICATIONS

Range of Motion Protection, Wrist Instabilities, Mild Strain and Sprains, Tendonitis.

CONTRAINDICATIONS

N/A

WARNINGS AND CAUTIONS

Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION

- Unfasten straps
- Slide hand into sleeve through the rounded side of the foam pads
- Grasp wide strap and tighten to desired fit
- Grasp narrow strap and tighten to desired fit

CLEANING INSTRUCTIONS

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS

86% Neoprene, 1% Nylon, 8% Silicone, 5% Polyester

LEVEL OF SUPPORT

INTENDED FOR SINGLE PATIENT USE

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE -OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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